

## BVD TESTING INSTRUCTIONS

### Preparation:

1. Prepare three clean liquid containers. Fill two containers with clean water. Fill the third container with properly diluted Nolvasan solution: 1 oz (30 ml) of Nolvasan per gallon of water.
2. Number sample vials with a permanent marker to match ear tag numbers.



### Testing:

1. Dip notcher in water; then dip in Nolvasan to disinfect; then in second container of water to rinse. Use containers in the same order each time.
2. Take a full notch from the top of the ear with the ear notcher provided.



3. Place the notch in the sample vial.
4. Dip notcher in water; then dip in Nolvasan; then in second container of water to rinse.
5. Check vial and ear tag to confirm the numbers match.
6. After samples have been notched from all cattle, ship samples to the lab or refrigerate until samples are shipped.